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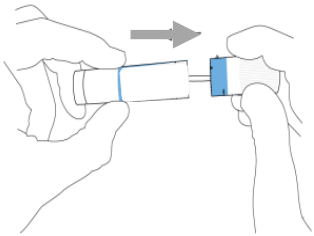
A SAMPLE TAKING FOR A FORENSIC GENETIC PARENTHOOD TEST UNDER CHILD SUPERVISOR'S SUPERVISION

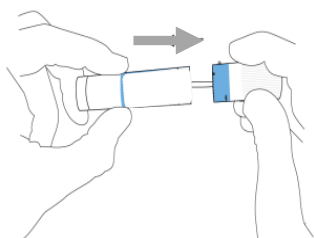
- Child supervisor enters the sample collection information in *Document of sample taking for forensic genetic parenthood test (VSAK-15) (Finnish-English)*.
- Before taking the samples person to be tested has to prove their identity by a valid document which can be driving licence (Finland), passport (all countries), identity card (EEA countries, Switzerland, San Marino), refugee travel document (Finland) or alien's passport (Finland) (Act 378/2005, section 13 a; Decree 168/2024, section 2).
- In certain situations, samples can be taken even if the identity of the person to be tested could not be verified in a reliable manner (Act 775/2022, section 29(2); Act 378/2005, section 13 a).
- **The person to be tested takes their own samples from the inside of the cheek.** From a child, the samples are taken by the mother but can be taken by another adult, if necessary.
- The person to be tested must avoid eating, drinking (other than water) and smoking for at least half an hour before taking the sample.
- Perform steps 1–5 before opening the sample tube for the next person to be tested.

1 Child supervisor writes clearly with a ballpoint pen or a permanent marker the name and personal identity code of the person to be tested on two label stickers (two samples/person). The blue-rimmed labels are for the man, the red-rimmed labels for the mother, and the yellow-rimmed labels for the child.

2 Child supervisor opens two foil-wrapped tube packages and sticks the labels on the white cap of the tubes before the sample is taken.

3 The person to be tested opens one sample tube by pulling the white cap off (the cap is firmly on). Avoid touching the swab tip against any surface.

4 The person to be tested firmly rubs or rotates the swab along the inside of the cheek for about 10 seconds and puts the swab immediately back to the tube and closes the tube tightly. Be careful not to press the swab so hard that the shaft breaks (this kind of situation is unlikely).

5 The person to be tested repeats steps 3 and 4 but takes the sample from the other cheek using another swab.

6 The sample tubes, the document of sample taking and the order document are put into the return envelope (no stamp is needed). All the samples which are taken during the same visit for the same investigation of parenthood can be packed into the same envelope.

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